GOOD POSTURE STARTS HERE

USER MANUAL

MANUEL D'UTILISATION • MANUALE DELL'UTENTE • MANUAL DE USUARIO • GEBRAUCHSANWEISUNG

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DEUTSCH

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HOW TO GET STARTED

1. Download the **UPRIGHT** app.
2. Follow the tutorial in the app for a step-by-step guide on how to get started.

TIPS FOR SUCCESS

- For best results within **2 weeks**, it is highly recommended to **use the GO S daily**
- Use your GO S in a cool **indoor** environment
- Leave your GO S **on your desk**, in its case, as a reminder to train
- Finally, don't forget to **charge** your GO S

DEVICE SHORTCUTS

Turn On

Short Click

Turn Off

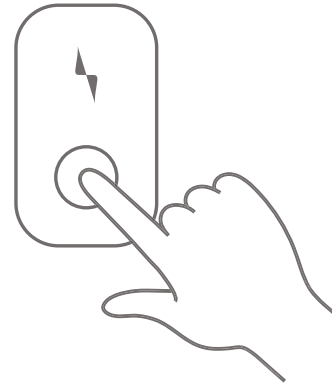
Long Click

Set Posture (Calibration)*

Double Click

Switching Modes Training/Tracking*

One Click

* Learn more about Set Posture, Training & Tracking in the UPRIGHT app

WHAT'S IN THE BOX

UPRIGHT GO S

Your personal posture trainer, measures your posture 100 times a second

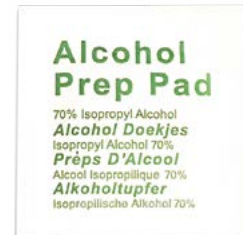
CHARGING CABLE

A USB-C cable for charging your GO S

CASE

Store your GO S in the case while charging it on your desk or when you're on the go

ALCOHOL PADS

Use the alcohol pads to clean the adhesives and/or your skin before use

ADHESIVES

Reusable silicone adhesives for attaching the GO S to your upper back

ADHESIVE REMOVAL TOOL

Use it to peel off your old adhesive when replacing it with a new one

CHARGING YOUR GO S

Charging your battery regularly will allow you to use your GO S as often as you choose.

- Place your GO S in its case for safekeeping and plug the charging cable into the USB port.
- A full charge takes about 3 hours.
- Once your battery is fully charged, the LED light will turn green.

LED INDICATORS

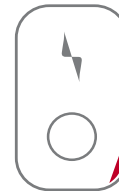
GREEN

Connected
to the app

BLUE

Not
connected

RED

Battery needs
charging

ADHESIVES

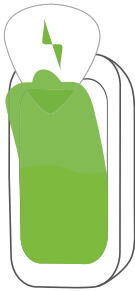
CARE & MAINTENANCE

- With proper maintenance, each adhesive can be used about **3-10 times**, depending on skin type.
- Make sure your skin is **clean and dry** before using the GO S. Use the alcohol pads provided to clean skin.
- The adhesive doesn't attach well to sweaty skin. Therefore, we recommend **training indoors**.
- When you're not using your GO S, **store it in its case** to keep the adhesive clean.

REPLACING THE ADHESIVE

1. Remove the old adhesive from the GO S.

We recommend using the removal tool provided.

2. **Important:**


Make sure you remove ALL parts of the old adhesive completely, so there's nothing green left on your GO S.

3. Take a new adhesive and peel off just the white cover from one side.

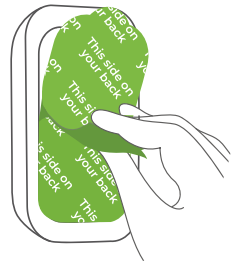
4. Attach the sticky part to the GO S, inside the thin gray frame.

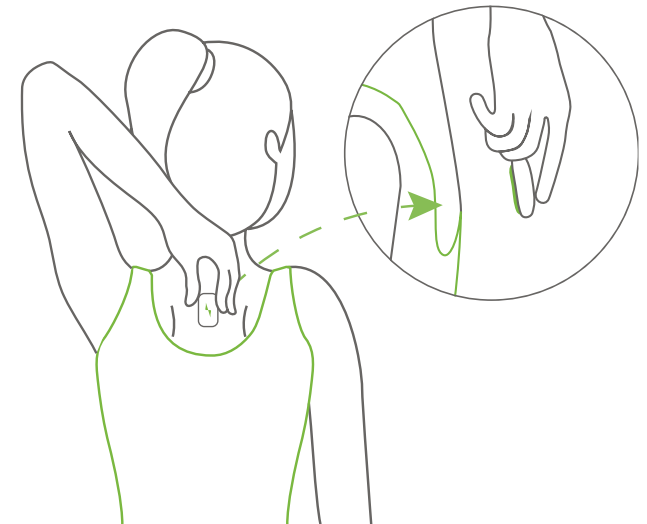5. When you want to start using your GO S, peel off the green cover that says "This side on your back".

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